

Codman VersaTru® Disposable Non-Stick Bipolar Forceps

Excellent Non-Stick Performance From Start to Finish.*

Testing was conducted comparing the non-stick performance of Codman® Non-Stick Forceps vs a leading competitor during continuous coagulation. This test was conducted at a high-intensity setting.*



80
COAGULATION
EVENTS

Codman Non-Stick Forceps



<5
COAGULATION
EVENTS

Leading Competitor

Maximize surgical performance in challenging situations*

- Codman's full length, proprietary plating optimizes non-stick capability
- Designed to reduce sticking and charring while coagulating
- Excellent non-stick performance, especially under intense coagulation conditions*

*A power intensity of 80 Malis units (24 watts) for 10 seconds of activation at each test site.

Codman
SPECIALTY SURGICAL

Non-Stick Performance That Outlasts Leading Competitor

To compare tissue adhesion of forceps tips due to the coagulation procedure, side-by-side testing of the Codman Non-Stick Forceps and a leading competitor was conducted.

The testing utilized significant power intensity, long continuous duration, and high repetitions to challenge both the control and test samples with extreme heat generation.*

Failures were noted if there was tip discoloration or tissue buildup that could not be wiped off.

Acceptance Criteria	Codman Non-Stick Forceps (n=5) 80 Coagulation Events	Leading Competitor (n=5) <5 Coagulation Events
No Tip Discoloration	✓	✗
No Tissue Adherence	✓	✗


*A power intensity of 80 Malis units (24 watts) for 10 seconds of activation at each test site.

Order Information:

Codman VersaTru® Standard Disposable Non-Stick Bipolar Forceps

Code #	Overall Length	Tip Size
9007050ST	7" (18 cm)	0.5 mm
9007100ST	7" (18 cm)	1.0 mm
9007150ST	7" (18 cm)	1.5 mm
9008050ST	8" (20 cm)	0.5 mm
9008100ST	8" (20 cm)	1.0 mm
9008150ST	8" (20 cm)	1.5 mm
9009050ST	9" (23 cm)	0.5 mm
9009100ST	9" (23 cm)	1.0 mm
9009150ST	9" (23 cm)	1.5 mm

1 pair per box

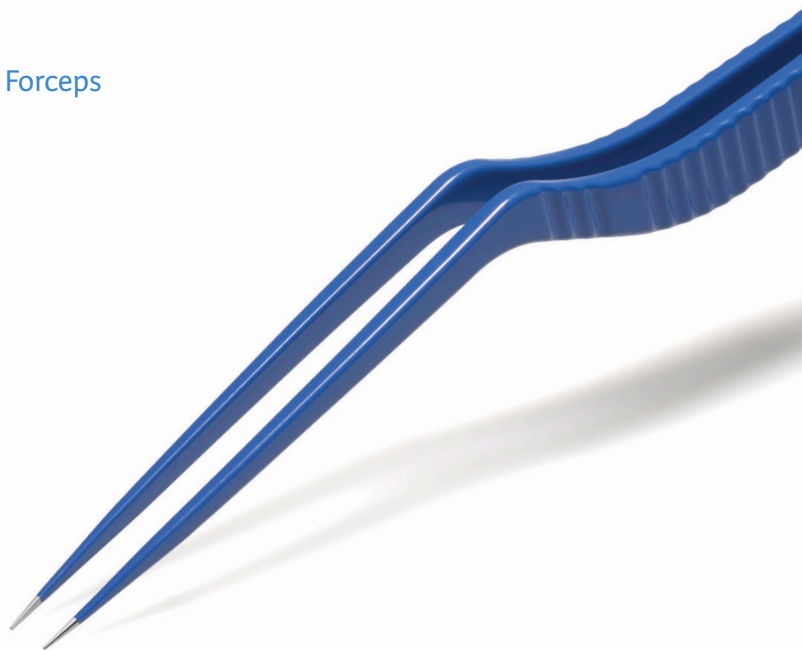

Codman VersaTru® Slim Disposable Non-Stick Bipolar Forceps

Code #	Overall Length	Tip Size
9008050SL	8" (20 cm)	0.5 mm
9008100SL	8" (20 cm)	1.0 mm
9008150SL	8" (20 cm)	1.5 mm

1 pair per box

Code #	Overall Length	Tip Size
9009050SL	9" (23 cm)	0.5 mm
9009100SL	9" (23 cm)	1.0 mm
9009150SL	9" (23 cm)	1.5 mm

1 pair per box

INDICATIONS FOR USE:

Codman VersaTru® Disposable Non-Stick Bipolar Forceps are intended for use in electrosurgery for coagulation of tissue.

CONTRAINDICATIONS:

- Resterilization of used product by the customer
- Use for contraceptive coagulation of fallopian tube tissue

Please consult the Instructions for Use for more detailed information about Warnings, Precautions, and appropriate use of the devices.

For more information, contact your Codman Specialty Surgical Account Manager.

Codman and Codman VersaTru are registered trademarks of Integra LifeSciences Corporation or its subsidiaries in the United States and/or other countries. ©2018 Integra LifeSciences Corporation. All Rights Reserved. OM 0884252-1-EN

Codman®
SPECIALTY SURGICAL